Effect. **TWiNY**
TWiNY Flow

INDEX

Young Anterior Application	03
Cervical/white Effect	04
Internal Effect/Enamel	05
Completion	06
Old Anterior Application	07
Cervical · Internal Effect	08
Internal Effect/Enamel	09
Completion	10
Premolar Application	11
Cervical/Occlusal Effect	12
Occlusal · White Effect	13
Completion	14
Molar Application	15
Cervical/Occlusal Effect	16
White Effect	18
Completion	19
Inlay Application	21
Dentine	22
Occlusal Effect	23
Completion	25
Inlay Application with Enamel Clear	27
Occlusal Effect	28
Completion	29
Product Line-Up, Related Products	30
Product Line-Up	30
Related Products	31

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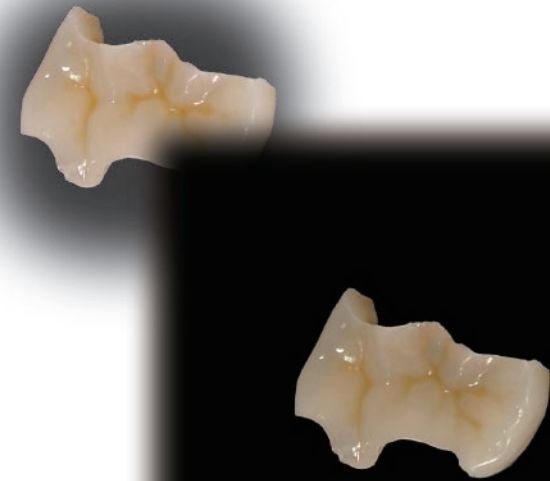
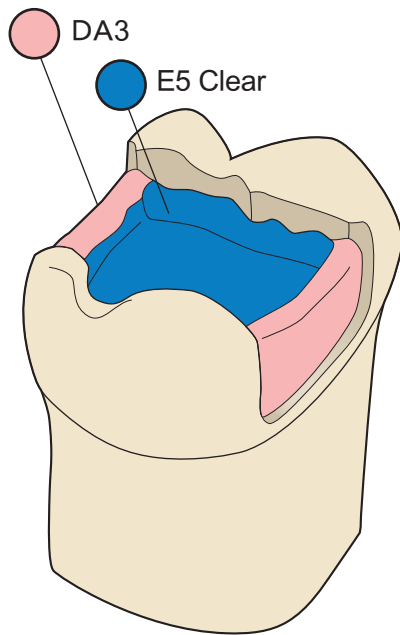
Effect.


TWiNY & TWiNY Flow

Inlay Application with Enamel Clear

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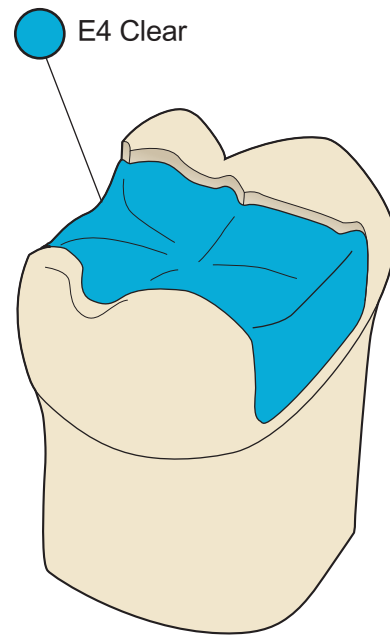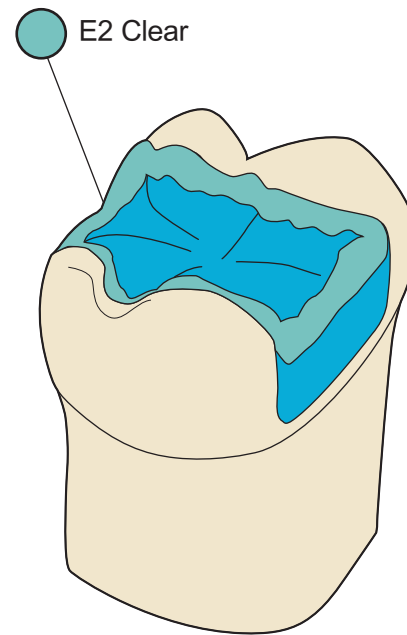
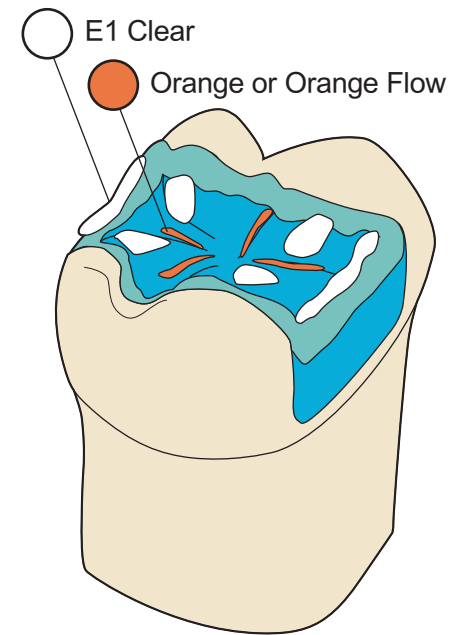
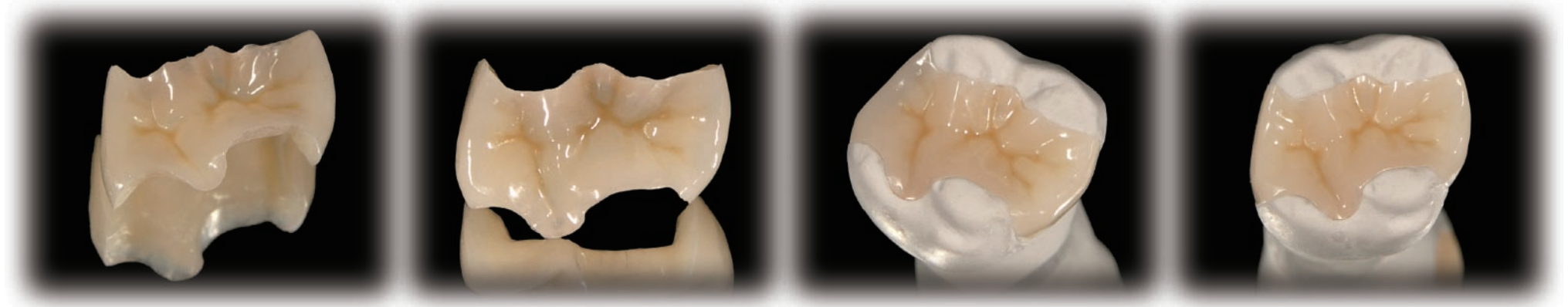
Note: Characterizing Opaque and Body resin to reproduce gum or express discolored gum, etc.

Gum Stain

GDarkRed	G Red	G Milky	G Violet
G D R	G Red	G Mky	G Vlt

TWiNY Flow


								
DA3 Flow	E3 Flow	CA1 Flow	CA2 Flow					
								
Coffee Flow	Orange Flow	AM Flow	OC Flow	WE Flow				
								
TE Flow	HVT Flow	T Flow	LVT Flow	CT2 Flow	CT4 Flow	T Glass Flow	T Glass Clear Flow	T Blue Flow
								
G1 Flow	G3 Flow	G5 Flow	G7 Flow	G Or Flow	GM Gray Flow	GM Trans Flow		

Luna-Wing Stain

								
White	Milky	Pink	Salmon Pink	White Violet	Violet	Orange	Dark Orange	Red
								
Yellow	Blue	Brown	Red Brown	Dark Brown	A Shift	B Shift	C Shift	D Shift
								
Gray	Black	Clear						

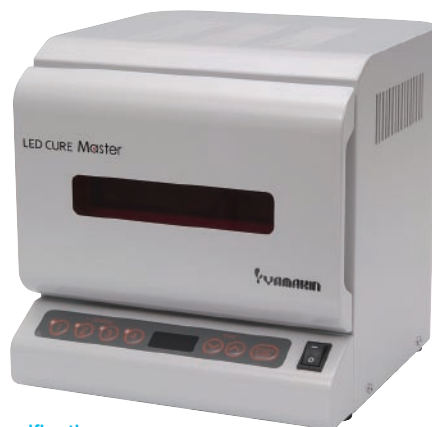Related Products

Optimal Light-Curing of Indirect Composite Resin

LED CURE Master

Curing Time for Luna-Wing and TWiNY

	LED CURE Master	conventional curing time
Primer Paste	10Sec.	90Sec.
Invisible Opaque	10Sec.	90Sec.
Opaque	30Sec.	180Sec.
Body (Dentine, Enamel, etc)	10Sec.	60Sec.
Base	90Sec.	180Sec.
Stain	10Sec.	60Sec.
Final Light Curing	90Sec.	180Sec.

Operations Course Buttons for Luna-Wing and TWiNY

	Dry time	Curing time	Course Button
Primer Paste	120Sec.	10Sec.	Operate manually
Invisible Opaque	—	10Sec.	①
Opaque	—	30Sec.	②
Bodyz (Dentine, Enamel, etc)	—	10Sec.	①
Base	—	90Sec.	③
Stain	—	10Sec.	①
Final Light Curing	Luna-Wing TWiNY	90Sec. 90Sec.	③ ④

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Specifications

Usage environment	Temperature 5-40°C, Humidity 10-95%RH, altitude max2000m
Operating Voltage and Frequency	AC230V±10% 50Hz
Power consumption	160VA
Number of course	4 Courses
Curing time	5-995 Sec.
Drying time	5-995 Sec. (Course button 4.)
Revolutions per minute	8.3r/min (50Hz)
External dimensions	210(W)×220(H)×223(D)mm
Dimensions of chamber	135(W)×77(H)×135(D)mm
Weight	5.4 kg
Emission wavelength	375nm ~ 495nm
Quantity of LED lamps	Power LED lamp 40pcs
Accessories	Tray: 1 pc, Big Pin: 5 pcs, Small Pin: 5 pcs

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Multi Primer Series

Multi Primer PASTE

Bonding Material for Dental Metal
NET: 2ml

Multi Primer LIQUID

Bonding Material for Dental Ceramics
NET: 7ml

Note: Flammable product

**Multi Primer
REPAIR LIQUID ONE**

Bonding Material for Dental Resin
NET: 6ml

C/ Maestrat 41-43 1º 08225 - Terrassa (Barcelona)
+34 93 780 47 99 - info@tonal.es
www.dentaltonal.com

Head Office: 3-7 Sanadayama-cho Tennoji-ku Osaka 543-0015, Japan
Branch Office: Tokyo, Osaka, Sendai, Nagoya, Fukuoka, JAPAN
Factory and R&D: Kochi, JAPAN
P: +81-6-6761-8338 F: +81-6-6761-4711
E: contact@yamakin-gold.co.jp

ISO 9001/13485
ISO 14001
CERTIFIED

Scope:
Headquarters and Kochi Plant

QAIC/JP/0455
Scope : Kochi Plant

INTERNATIONAL 20150611